Actimo® Arthro

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DE

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Actimo® Arthro**User manual**

EN

Read the entire user manual carefully before using the Actimo® Arthro for the first time. Keep the instructions for use so that they are available for later use.

Intended Purpose

The rollator Actimo® Arthro is used to support the walking of persons who are restricted in their mobility and mobility due to a rheumatic disease or an inflammation of the upper limbs and hands and who need a stable hold. The rollator has a seat as well as a removable bag for transporting shopping and utensils. The Actimo® Arthro is only suitable for use on level floors in domestic use.

Indication

The Actimo® Arthro can be used in cases of slight restriction of mobility of the legs and knee joints, slight loss of strength and slight balance disorders. The rollator Actimo® Arthro is suitable for maintaining physical activity and mobility. The high padded supports serve to promote stability and/or reduce pain when pushing the arms and hands.

Kontraindikation

The use of the rollator is not suitable for

- perceptual disturbances
- severe balance disorders
- severe pain in the trunk or lower limbs
- loss of strength
- reduced or insufficient vision
- Loss of limbs in both arms
- joint contractures in both arms
- and advanced arthritis of the hands and fingers due to inability to safely handle the brakes

Assembly

Your specialist dealer will be happy to help you with the installation.

1. Remove the pre-assembled rollator from the box and check the contents as well as the packaging for damage and completeness.
2. Unfold the frame of the rollator by pressing the seat down on both sides until you hear it click into place.

3. Insert the tubes with the arthritis supports into the frame parts of the stand. Loosen the knob screw on the tube support here, push or pull the handle tube to the desired height (holes for fixing) and tighten the knob screw again. Make sure that the same height is set on both sides. The brake cables run parallel to the frame.

4. To adjust the distance between the brake/handle and the arthritis support,

unscrew the nut below the support and remove the screw from above. Pull the handle out until the desired distance is reached and put the nut back into the opening from above. Push it all the way in and tighten the nut again from below. The distance should be the same on both sides.

5. Attach the bag to the suspension device and to the lower part of the frame using the Velcro fasteners.

6. To fold the rollator, remove the bag or empty it completely and pull the handle on the seat upwards.

Use of the brakes and parking brakes

1. Always use both brakes simultaneously.
2. To slow down or stop the rollator for a short time, brake by simultaneously pulling the brake levers upwards/ towards you until you have reached the desired speed or stopped. 3.
3. To lock the parking brakes, push the brake levers forward at the same time until they engage. 4.
4. 4. to release the parking brake, simultaneously pull the brake levers towards you/backwards until they disengage from the lock.
5. Make sure that the Bowden cables run in front of the rollator and are not twisted. 6.
6. The braking effect is adjusted on the brake lever or the Bowden cable. The brakes are correctly adjusted if the brake levers have no play and the wheels turn without grinding noises when the brake is released.

Safety instruction

- - Only use the rollator if it is in perfect condition. Do not use a defective rollator!
- - Before each use, make sure that the rollator is fully unfolded and the seat is locked in place.
- - Make sure that the handles are at the same height and test the stability of the rollator before carefully putting all your weight on it.
- - Test the brakes before each use and always use the right and left brakes at the same time to brake!
- - Pay attention to the wear of the tyres! Heavily worn tyres can impair the braking effect of the rollator. Have damaged parts replaced by your specialist dealer.
- - When the parking brakes are activated, the wheels must not turn, otherwise the brakes/parking brakes must be readjusted.
- - Activate the parking brakes before sitting on the seat. Only sit down when the rollator is on firm, level ground.
- A rollator is not a wheelchair, but only a walking aid. Do not allow yourself to be pushed when you have taken a seat.
- Only use the rollator on level ground - never on stairs or high edges! To ensure stability and avoid accidents, all four wheels must always be in contact with the ground during use.
- Always hold on to both handles to keep your balance.
- Before each use, check that the armrests are firmly in place!
- The rollator must not be carried by detachable parts or brake cables.
- Apply the brakes when folding and unfolding the rollator.
- Be careful when unfolding and folding the rollator. - There is a danger of jamming!
- Do not hang any bags on the handles or armrests. - There is a danger of tipping over!
- Do not overload the rollator (max. luggage load: 10 kg).
- The maximum user weight is 136 kg.
- Do not make any constructive changes to the rollator that you have not previously agreed with your specialist dealer or Rehaforum Medical GmbH!

Cleaning

Clean and disinfect the rollator at regular intervals, e.g. daily. Before passing the rollator on to another person, it must be thoroughly cleaned and disinfected. Use a mild soap solution without abrasive additives and a soft cloth. Disinfection can be carried out with commercially available agents according to the manufacturer's instructions. Do not use any abrasive or corrosive cleaning agents or high-pressure or steam cleaners! Make sure that no dirt is deposited on the tyres or brakes. This can be removed with a soft brush and warm water. Also make sure that no cleaning fluid runs into the frame tubes to avoid corrosion!

Maintenance

The rollator Actimo® Arthro should be serviced annually by your specialist dealer.

The stability of the frame, the strength of the mountable parts and the tightness of all screws must be checked and repaired if necessary.

All screws must be tightened. Worn, damaged or broken parts must be replaced immediately.

Malfunction:

If you notice a reduction in braking power, sluggishness of the wheels, uneven running of the wheels or other faults such as rust on the linkage, contact your specialist dealer. Do not use a defective or damaged rollator!

Re-use

The rollator is suitable for reuse. Before passing it on to another user, the rollator must be inspected, cleaned, disinfected and serviced by a specialist dealer.

If parts of the rollator are damaged or destroyed, they must be replaced with new ones. It is recommended that the wheels are also replaced each time the user changes the rollator. All loosened screws and nuts must be tightened.

The product must be in perfect condition before it is used again.

It is forbidden to reuse the rollator if irreparable damage to the rollator such as a bent frame is found!

The operating instructions must be handed over each time the user changes.

Scope of delivery

Actimo® Arthro

Shopping bag

User manual

Technical data

Weight with armrests without bag: 9.1 kg; with bag: 9.5 kg
Tube diameter (oval): Width approx. 1 cm, length: 4.5 cm
Seat width: 46 cm
Seat depth: 23 cm
Seat height: approx. 61 cm
Adjustable height of forearm rest: 102 - 114 cm (in 7 steps)
Distance of the arm rests to the handle: 7-19 cm (in 5 steps)
Total length of armrest frame: 38 - 51 cm
Length of the arm rests: approx. 17 cm
Width between the forearm rests: 50 cm
Total length: approx. 76 cm
Total width: 68.5 cm
Overall height: 112.5 - 118 cm
Turning circle: 168 cm
Tyres front: 10"
Tyres rear: 8"
Tyre width: approx. 3 cm
Materials frame: anodised aluminium
Material pads: PU (plastic)
Material handles and wheels: TPE (thermoplastic elastomers)
Maximum load of the bag: 10 kg
Maximum load capacity: 136 kg
Recommended body height: approx. 1.65 - 1.90 m

Storage

Protect the rollator from dust and moisture and store it in a safe place to protect the linkage and wheels from damage! Protect your rollator from strong sources of heat!

Disposal

Do not dispose of the rollator with normal household waste. Please contact your specialist dealer or take the rollator to your local disposal point.

Service life

If the rollator is used within its intended purpose and is cared for, checked and maintained according to the specifications, a service life of five years can be assumed for the Actimo® Arthro. However, the service life can vary considerably depending on the treatment and care of the product. For example, the service life can be extended by particularly careful handling of the product, while the service life can be shortened by extreme and improper handling of the product.

The determination of the service life by Rehaforum Medical GmbH does not constitute an additional guarantee.

Warranty

The warranty period is 24 months and applies to all product defects that can be proven to be due to manufacturing or material defects.

In particular, the warranty claim does not apply if the rollator has features that indicate repairs or other interventions by unauthorised specialist dealers, or if third-party accessories have been installed in the rollator without the prior written consent of Rehaforum Medical GmbH.

Notification / Incident

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or the patient is established.

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Rehaforum Medical GmbH's bestemmelse af levetiden udgør ikke en yderligere garanti.
Rehaforum Medical GmbH, Daimlerstraße 12a, DE-25337 Elmshorn