Toilettenstuhl RFM® fahrbar

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DE

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User manual

EN

Please read the entire user manual carefully before assembling, adjusting, or using the product for the first time. Keep the user manual for future reference.

### Purpose

The commode chair is intended for hygienic toilet use and to facilitate personal care for individuals who require assistance with mobility and/or are in need of care.

### Indication

The commode chair can be used for mild balance disorders, lower limb functional impairments, paralysis, lack of trunk control, and/or limited mobility of the hip and knee joints.

### Contraindication

The use of the commode chair is not recommended for individuals with:

- Sensory disorders
- Severe balance disorders
- Amputated limbs

### Scope of Delivery

Commode chair preassembled

Toilet bucket with lid

Seat

User manual

## Assembly

Please have this assembly performed by a specialist dealer/nursing staff.

1. Carefully open the box and remove the commode chair. Lift off the backrest and place it in front of you on the floor or table.
2. Remove the wheels from the accessory parts box. Remove the bucket and lid from the frame and turn it upside down, with the pipe openings facing up.
3. Insert a wheel into each opening and secure it with a screw and nut. Ensure that the rear wheels are inserted into the rear pipes with brake levers. These are located where there is only one horizontally running pipe.
4. Place the commode chair on the wheels and insert the footrests with the overlapping openings from above onto the two pins located on the left and right side of the frame. Fold them inward using the black lever. Secure the strap to the footrests with the help of the Velcro fasteners.
5. To fold the footrests outward, gently push the black levers away from the frame. The footrests can be immediately swung outward.
6. Slide the backrest from the top into the open ends of the frame and tighten it with the two large knob screws.
7. Put the armrests in a horizontal position and secure them by pulling the tension screws backward and then gently sliding them into the openings. To fold down the armrests, pull the tension screws backward until the lock is released.
8. Place the frame with the notch on the commode chair's frame and fold it down. The wide opening of the guide rail, located beneath the frame, should face backward. Slide the bucket with lid from behind into the guide rail. Two resistors located in the guide rail will secure the bucket for safe transport. To empty the bucket, remove it simultaneously with the lid.
9. To use the commode chair as a seat, place the seat on the frame.

## Brake Usage

There are locking brakes on the two rear wheels, which are engaged by pushing down the larger levers. Pushing down a smaller lever will release the brakes. Directional locks are also located on the rear wheels, which are activated by engaging the brakes to prevent the commode chair from moving.

### Safety Instructions

- Before each use, check if the commode chair is in good condition and ensure that all connections are securely fastened.
- Check the fit and tightness of all screws/tension screws to ensure the safe use of the commode chair.
- To ensure the best service of your commode chair, always use original spare parts available from your dealer.
- Release the footrests gently using the black levers to avoid jamming.
- Ensure that the patient has placed their arms and legs on the armrests or footrests to prevent injuries during pushing.
- Make sure the patient has not placed their arms on the armrests before releasing them for folding.
- Sitting on the armrests is strictly prohibited!
- Press both levers on the lockable casters downward before using the commode chair to prevent unintentional movement.
- Avoid leaning too far forward and making strong jerking movements.
- Never step on the footrests when getting on or off the commode chair!
- The use of the commode chair is strictly for indoor use.
- Use the commode chair only on a horizontal, flat, and firm surface.
- Do not use a defective commode chair!
- The maximum user weight is 130 kg.

### Cleaning

Clean and disinfect the product at regular intervals, e.g., daily. Before handing the product to another person, thorough cleaning and disinfection are mandatory. Use a mild soapy solution without abrasive additives and a soft cloth. Disinfection can be done using commercial products according to manufacturer instructions. Do not use abrasive or corrosive cleaning agents, high-pressure, or steam cleaners. Ensure that no dirt accumulates on the wheels or brakes, which can be removed with a soft brush and warm water.

### Maintenance

The RFM mobile commode chair is maintenance-free. All padded surfaces and the bucket are made of easy-care PE or PVC, which is water-repellent. The frame is made of powder-coated steel, making it resistant to corrosion.

### Malfunction:

If you notice any damage, wear and tear, or loose components, contact your dealer. Do not use a defective commode chair!

### Reuse

The commode chair is suitable for reuse. Before handing it over to another user, the commode chair must undergo inspection, cleaning, disinfection, and maintenance by the dealer.

If parts of the commode chair are damaged or destroyed, they must be replaced with new ones. All loosened screws and nuts must be tightened. The product must be in perfect condition before reuse, and the user manual should be provided to each new user.

Reusing the commode chair is prohibited if irreparable damage, such as a bent frame, is observed!

### **Technical Data**

Weight: 13.8 kg  
Seat height: 53 cm  
Total height: 95 cm  
Backrest height: 19 cm  
Seat width: 45 cm  
Seat depth: 44 cm  
Total depth: 77 cm  
Seat angle: 90° with closed cover  
Armrest height: 30 cm  
Width between armrests: 45 cm  
Backrest width: 44 cm  
Total width: 57 cm  
Backrest padding: 15 mm  
Armrest padding: 30 mm  
Wheel diameter: 12 cm  
Maximum load: 130 kg

### **Storage**

Protect the commode chair from dust and moisture, and store it in a safe place to prevent damage to the frame and wheels. Keep your commode chair away from strong heat sources.

### **Disposal**

Do not dispose of the product in regular household waste. Please contact your dealer or take the product to your local disposal facility.

### **Lifespan**

If the product is used within its intended purpose and maintained, checked, and serviced as specified, a lifespan of five years can be expected. However, the lifespan can vary significantly depending on the treatment and care of the product. Careful handling can extend the lifespan, while extreme and improper handling can shorten it. The manufacturer's determination of the lifespan does not provide additional warranty.

### **Warranty**

The warranty period is 24 months and refers to all product defects that can be proven to be due to manufacturing or material defects.

In particular, the guarantee does not apply if the product has features that suggest repairs or other interventions by unauthorized specialist dealers, or if third-party accessories have been installed without prior written consent from the manufacturer.

### **Reporting/Incident**

All serious incidents related to the product must be reported to the manufacturer and the relevant authority in the member state where the user and/or patient is located.

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Rehaforum Medical GmbH, Daimlerstraße 12a, DE-25337 Elmshorn