Starlight

Super Lightweight Carbon Rollator

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1. General Information

Dear Customer,

Thank you for purchasing our medical device. In this user's manual, you will find all the important information related to the proper and safe use of this product, hereinafter referred to as the "rollator." Before using the rollator for the first time, please read and carefully familiarise yourself with the contents of this manual and follow all our recommendations.

It is especially important to read this user manual carefully before using the Starlight/Starlight+ rollator for the first time. It describes all the important features of our medical device.

To ensure correct and safe use of the product, it is necessary to become familiar with the device before use. This includes personal consultation and instruction by qualified specialist personnel in a certified medical supply store, such as a medical equipment retailer.

It is particularly important that you read and follow the safety requirements.

Rehasense continuously improves its products and reserves the right to make changes to the product specifications without prior notice.

2. Intended Use, Indications, and Contraindications

Indications: The rollator has been designed as a mobility aid for individuals who experience difficulties with standing and walking. It enables such individuals to return to many everyday activities.

It is intended for people who, due to various medical conditions, require stable support and assistance in moving around. The rollator must not be used for purposes other than providing support for standing, walking, and sitting.

Contraindications: The rollator should not be used by individuals who have serious balance problems or significant weakness of the skeletal muscles that prevents them from standing or walking independently. The use of the rollator should be recommended by a physiotherapist or a doctor.

3. Delivery and General Product Description

The rollators are delivered in cardboard packaging in a folded position. The package contains:

- ✓ folded rollator
- ✓ crutch holder set

- ✓ user manual
- ✓ bakset

The Starlight rollator is equipped with 4 wheels. The lightweight carbon fibre frame allows it to be folded, stored, and transported.

The product offers easy manoeuvrability, stability, and adjustable handle height.

It is also factory-fitted with a seat and a shopping basket.

4. Requirements and Safety

The rollator may only be used by persons who have received personal consultation and instruction from qualified specialist personnel in a professional store, such as a medical supply retailer, and who have fully read and understood this user manual. The manufacturer assumes no liability for damages and/or injuries caused by improper use.

Before using the rollator, read the contents of this manual. The manufacturer is not responsible for any damage or accidents resulting from improper use.

Limitations for usage

- The user's weight must not exceed 150 kg.
- The standard shopping bag can hold up to 5 kg.
- The rollator is intended for use both indoors and outdoors.
- The rollator must not be used or pushed while seated.
- Before using the seat, the brakes **MUST** be in the locked position.

Accident Prevention

- Use the device only as intended.
- Always check that the brakes are working before using the rollator.
- Make sure that the folding mechanism is locked in place after unfolding the rollator.
- Always use the parking brake when stopping or parking.
- Ensure that the rollator has been adjusted to the correct height to avoid incorrect loadings and falls. Do not push the rollator a long way in front of your body.
- Be aware of the possibility of trapping injuries; avoid putting your fingers between the rollator's moving parts. Pay particular attention when folding and unfolding the rollator.
- Be aware that some parts of your rollator may be very hot or cold due to ambient temperature, solar radiation, heating devices, etc. Be careful when touching.
- Do not hold the handgrips with wet hands. If you do so, you can

- lose your grip and lose your balance.
- Do not proceed with your rollator into traffic. Always stay on the pavement.
- If the rollator is subject to direct sunlight / cold temperature for a long period of time, then the surfaces of the rollator may become hot ($>41^{\circ}\text{C}$) / very cold. Be careful when touching the rollator in these conditions as skin burns or freeze can occur. People with Hypoesthesia should be extra careful.
- Users must move among pedestrians and not on trafficked roadways. Use reflectors when using the rollator outdoors.
- Do not use the rollator for any purpose other than that it was intended for. It must not be used to transport people or heavy loads.

IF ANY SERIOUS INCIDENT OCCURS IN RELATION TO THE DEVICE, PLEASE CONTACT YOUR DEALER OR REHASENSE DIRECTLY (contact details at the end of the user manual). The immediate actions should be taken according to the requirements of Regulation of the European Parliament and of the EU Council 2017/745 on medical devices of April 5, 2017.

 Mind your fingers	Be careful when you see this symbol on the device. Fingers or other body parts of a user and/or assistant can be trapped between the device's parts during folding, unfolding or adjusting of the device. Be extremely careful when adjusting the device. Observe the dimensions of gaps between the device's elements to avoid trapping your fingers or other body parts.
---	---

5. Device Description

Pic.2

The photo above shows an example rollator with all standard components and their positioning. The exact appearance of the rollator and its functions may differ from the rollator shown above (depending on the order specifications).

- Ⓐ Push handles
- Ⓑ Brake lever
- Ⓒ Seat
- Ⓓ Bag
- Ⓔ Brake cable
- Ⓕ Side frame
- Ⓖ Cross brace
- Ⓗ Brake cover
- Ⓘ Height adjustment mechanism
- Ⓢ Folding handle
- Ⓚ Wheel fork
- Ⓛ Carrying handle and bag holder
- Ⓜ Backrest

Reflectors:

6. Use

The Pic. 2 shows the Starlight rollator along with all accessories. After receiving the rollator, follow the instructions below:

Initial Inspection

After removing the device from the packaging, first check that nothing is missing and that all parts shown in the pic.2 are included in the box. If you notice any missing parts, contact the device supplier immediately.

Unfolding the rollator

- Push the red button which is locking the seating tubes
- Hold both seat tubes and simultaneously push them down until you hear a “click” sound of the locking mechanism of the frame. Try to pull the seat tubes back up to fold the frame. If this is impossible to do the frame is opened and secured correctly. See the OPEN position in the pic.2.

- Always ensure, when unfolded, that the strap attached to the seat is secured; it facilitates folding the rollator.
- Always check before use that all parts of the rollator are in proper working condition.
- Failure to follow these instructions may result in injury to the user.

Folding the Rollator

- Lift the pulling strip on the seat to fold the rollator.
- Fold the rollator until the lock closes.

Adjusting Handle Height

- Adjustment is done by pulling upward the handle height adjustment mechanism.
- Pull one handle, then the other, upward so they are at the same height.
- Both handles should always be set to the same height.
- Never leave the device with unlocked brakes.
- Correctly positioned handles should be at the user’s hip level.

Braking / Releasing the Brakes

- To stop the rollator, pull the brake handles upward.
- To lock the wheels for an extended period, press the brake handles downward until you hear a “click.”
- Only with the brakes locked may the user turn around and sit on the seat.
- To move again, stand up, turn, and release the brakes by lifting the handles upward.

Safety Rules for Brakes and Seating

- Always keep the brakes locked when the rollator is stationary.
- Never sit on the seat without locking the brakes!
- Never lean backward while seated. It is recommended to sit at the front of the seat, taking care not to sit “on” the bag.
- Sit upright and hold the handles.
- Never leave the rollator unlocked on slopes or inclines!

Seat Use

- Before use, make sure the brakes are locked.
- Always sit with your back to the direction of movement.

7. Cleaning, Disinfection, and Maintenance

General:

- The rollator and its accessories should be cleaned and maintained regularly to ensure optimal performance.
- Light dirt and dust can be removed with a damp cloth.
- Heavier dirt can be cleaned with a mild, warm soap solution.
- Do not use abrasive cleaners or bleach.
- Do not clean with solvents such as gasoline or thinner, as these can damage the paint and plastic components.
- Clean the wheels with warm water and a mild detergent.
- The rollator and accessories can be cleaned in a washing cabin at a maximum temperature of 80°C for up to 10 minutes.
- When using high-pressure cleaning, do not direct the jet directly at the wheel bearings.
- The rollator and accessories can be disinfected using a soft cloth or sponge soaked in a disinfectant containing 70–80% ethanol. Disinfectants based on chlorine or phenol are not allowed.
- Regularly check that hair or other debris does not accumulate between the wheels and forks.
- Use only disinfectants approved for use (the current list is available on the Robert Koch Institute website: www.rki.de).

Wheels:

- Clean the wheels with warm water and a mild detergent. Dry them with a clean cloth.

8. Sample Label

The product label is located on the main frame, and the full description is on the penultimate page of the user manual.

9. Technical Data

Size	L	XL
Maximum user weight	150kg	150kg
Maximum bag load	5kg	5kg
Total weight without bag	6kg	6kg
Basket weight	0,43kg	0,43kg
Length	72cm	72cm
Height (unfolded)	78-97cm	86-105cm
Width (folded)	22cm	22cm
Width (unfolded)	60cm	60cm
Seat width	46cm	46cm
Handle height (adjustable)	79-98cm	83,5- 105kg
Seat height	63cm	67cm
Brakes	2x maneuver brakes with parking	2x maneuver brakes with
Wheels	Front 250x34mm Rear 200x34mm Black TPE tire	Front 250x34mm Rear 200x34mm Black TPE tire
Recommended user Height	150-185cm	165-200cm

10. Transport

The rollator is a foldable product, convenient for transport. It is recommended to transport it in the folded position, as described above. During transport, ensure the product is in a stable position and cannot move around.

11. Regular Inspection

- Check that the brakes function correctly. Adjust if necessary.
- Check that the wheels are in good condition and that they have not been damaged.
- Ensure that the rollator is clean and that all moving parts function correctly.
- Check and if necessary adjust all screwed connections.

12. Reuse

The rollator is suitable for reuse. Before being passed on to another user, it must undergo a technical inspection at an authorised service centre.

Inspection points:

- Visual inspection of the frame
- Secure fastening of nuts and bolts
- Condition of plastic parts
- Condition of wheels and ball bearings
- Effectiveness and condition of the brakes

If any part is damaged or broken, it must be replaced with a new one. It is recommended to replace the wheels in every case of user change. All loose screws and nuts must be tightened. Before reuse, the product must be thoroughly cleaned and disinfected.

Warning! The rollator must not be reused if the frame is bent or broken. The user manual must always be provided with the product when changing users.

13. Disposal

At the end of its service life, the device should be handed over for recycling to a company specializing in disposal.

14. Warranty

The warranty does not cover improper use of the product. It excludes damage resulting from incorrect operation, accidents, and/or normal wear and tear. Any modifications to the product will void the warranty. Mechanical damage is not covered by this warranty.

For warranty repairs, please contact your local dealer.

Warranty conditions:

1. Should a part or parts of the product require repair or replacement as a result of a manufacturing and/or material fault within the warranty period, then the affected part or parts will be repaired or replaced free of charge. The warranty will only cover manufacturing defects.
2. To enforce the warranty, please contact Rehasense (warranty@rehasense.com) with the exact details of the nature of the difficulty.
3. For parts which have been repaired or exchanged within the scope of this warranty, we provide a warranty in accordance with these warranty conditions for the remaining warranty period for the product in accordance with point 1).
4. For original spare parts which have been fitted at the customer's expense, these will have a 24 months guarantee (following the fitting), in accordance with these warranty conditions.
5. Claims from this warranty shall not arise if a repair or replacement of a product or a part is required for the following reasons:

- a. Normal wear and tear, which includes but is not limited to the following parts where fitted: batteries, armrest pads, upholstery, tyres, brake shoes, ferrules, etc.
- b. Any overloading of the product, please check the EC label for the maximum user weight.
- c. The product or part has not been maintained or serviced in accordance with the manufacturer's recommendations as shown in the user instructions and/or the service instructions.
- d. Accessories have been used which are not specified as original accessories.
- e. The product or part having been damaged by neglect, accident or improper use.
- f. Changes/modifications have been made to the product or parts, which deviate from the manufacturer's specifications.
- g. Repairs have been carried out, before our Customer Service has been informed of the circumstances.

6. This guarantee is subject to the law of the country in which the product was purchased from Rehasens.

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